



NxGen Pocket Spirometer Setup Instructions

Step 01 Insert Batteries

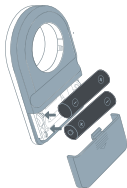

Open battery latch and insert AAA batteries. Close the latch securely until it clicks.

Step 02 Insert Turbine

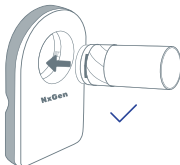

Insert the Safety Turbine from the front, as shown in the figure.

Step 03 Fit the Turbine in the Device



Gently push and rotate the Turbine in Clockwise direction until it is fully fitted.


Step 04 Download the App



Scan QR Code to Download the App.



Step 05 Setup Profile

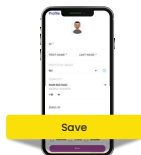


Tap "Continue" to Set Up Profile.

Scan Here To Download User Manual



Step 06 Allow Location Access

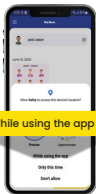

Allow for Location Access.

Step 07 Connect to the device



Tap "Connect" to Connect Device.

Step 08 Select Test Type



Select the type of tests and Tap "Next".

Step 09 Initiate Test



Check Battery Status, then Tap "Start Test".

Step 10 Test Procedure



Stand, hold the device, Inhale and exhale forcefully.

Step 11 View Results and Finish Test



View Graph and Results, then Tap "Finish".